



Aud shortcuts

Key Function

ArrowUp	Stimulus level up ^(a,b)
Arrow Dn	Stimulus level down ^(a,b)
Arrow Left	Frequency Down ^(c)
Arrow Right	Frequency Up ^(c)
SpaceBar	Stimulus on (or Off in case of inverse mode)
Alt / AltGr	Masking/Stim 2 on (or Off in case of inverse mode)
N	Stimulus Continuous on/off
M	Masking/Stim 2 Continuous on/off
S	Store current point
	Holding down the S key toggles between storing symbol, – no response symbol (symbol with arrow down) – response at limit symbol (symbol with arrow up, and – no symbol displayed

(a) Operation will depend on "Level direction of arrow key" – Tools>Options
 (b) Shift + key = Masking/Stim 2 level
 (c) Frequency wrap around at the edge

Key Function

L	Left routing on Stimulus ^(a)
R	Right routing on Stimulus ^(a)
B	Binaural routing on Stimulus ^(a)
F5	Speech player Play/Stop
T	Threshold or SRT
C	MCL
U	UCL
D	SDT
W	WRS/SRS

Numpad

1-9	Phoneme/Sentence Scoring
+ / -	Wordscoring

(a) Contralateral masking. If the stimulus channel routing is changed, the masking channel switches automatically to maintain the relative routing.

Stimulus selections

Tone	Speech
1 = Tone	1 = Mic (Live voice testing)
2 = Warble	2 = Source A (recording)
	3 = Source B (recording)

Communications

F/F2	Talk Forward ^(a)
P	Talk Back Level ^(b)

(a) Press and hold activates Talk Forward. Left/Right arrows then adjusts output level
 (b) Left/Right arrows then adjusts output level

Other shortcuts

General

New session	Ctrl+N
Patient Details	Ctrl+D
Open	Ctrl+O
Save	Ctrl+S
Print	Ctrl+P
Exit	Alt+F4
Previous Test Tab	PgUp
Next Test Tab	PgDn
Aud Properties	Ctrl+U
Toggle Stimulus Ear	Ctrl+E
Options	Ctrl+M
Test Selector	F8

PMM

Fitting Details	F10
Start Sequence	F5