


# Keyboard short cuts

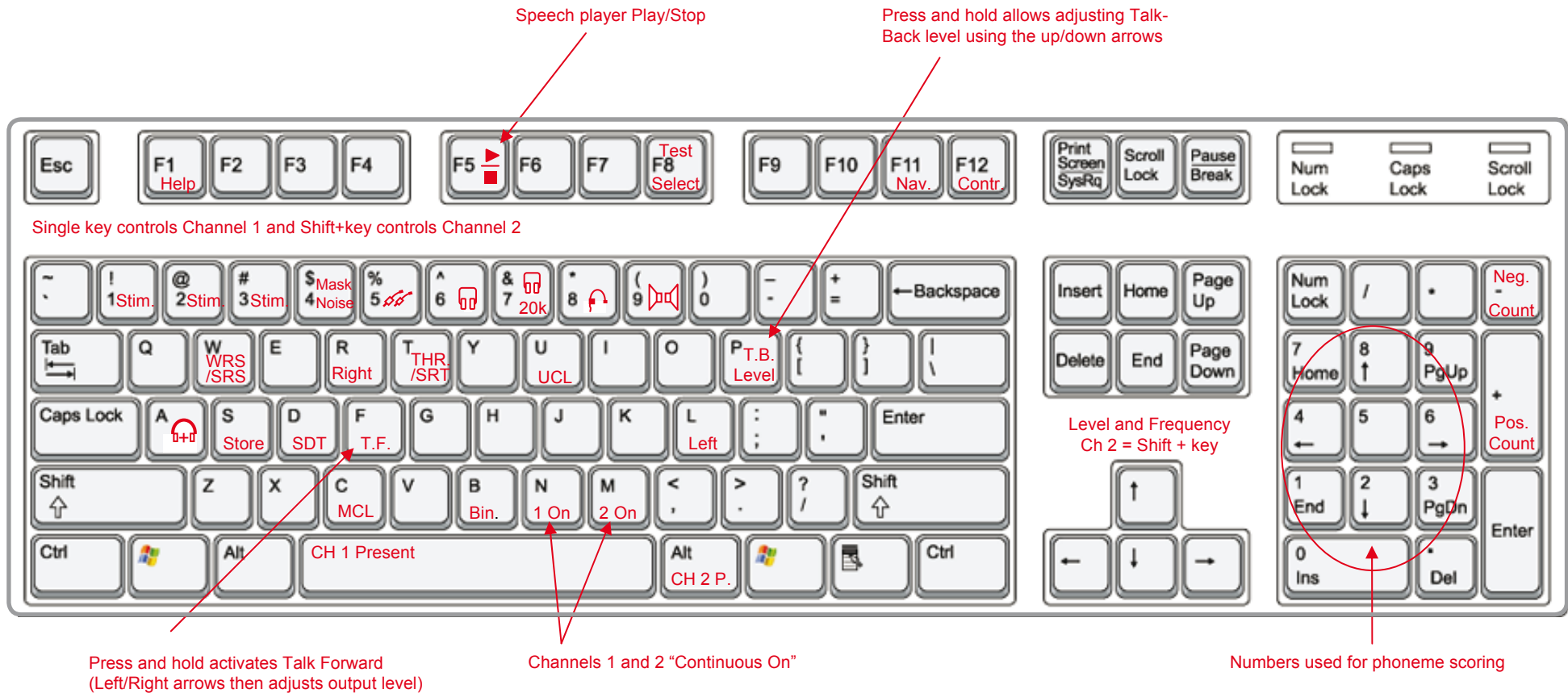

**Other short cuts:**

New Session	Ctrl+N	Previous Test Tab	PgUp
Patient Details	Ctrl+D	Next Test Tab	PgDn
Open	Ctrl+O	Aud Properties	Ctrl+U
Save	Ctrl+S	Toggle Stimulus Ear	Ctrl+E
Print	Ctrl+P	Options	Ctrl+M
Exit	Alt+F4	Navigator	F11
		Control Panel	F12

**Stimulus selections:**

<b>Tone</b>	<b>Speech</b>
1 = Tone	1 = Mic (Live voice testing)
2 = Warble	2 = Source A (recording)
3 = Narrow Band Stimulus	3 = Source B (recording)

### Key Function:

Arrow Up	Channel 1 level up <sup>(see a)</sup>
Arrow Dn	Channel 1 level down <sup>(see a)</sup>
Arrow Left	Frequency Down <sup>(see b)</sup>
Arrow Right	Frequency Up <sup>(see b)</sup>
SpaceBar	Channel 1 on (or Off in case of inverse mode)
N	Channel 1 Continuous on/off
M	Channel 2 Continuous on/off (Masking)
S	Store current point
	Holding down the S-key toggles between storing symbol,
	• no response symbol (symbol with arrow down),
	• response at limit symbol (symbol with arrow up), and
	• no symbol displayed
Shift + S	For storing channel 2 (when applicable)
NumPad 0-9	Phoneme Scoring

### Key Function:

L	Left routing on Channel 1 <sup>(see c)</sup>
R	Right routing on Channel 1 <sup>(see c)</sup>
B	Binaural routing on Channel 1 <sup>(see c)</sup>
F8	Opens the Test Selector
T	Threshold or SRT
C	MCL
U	UCL
D	SDT
W	WRS/SRS
A	Talk to Assistant
F	Talk Forward (←→ sets T.F. level)
F5	Speech player Play/Stop
P + ↑ ↓	Talk Back level

(a) Operation will depend on "Level direction of arrow key"  
– Tools>Options>Tone>Mic

(b) Frequency wrap around at the edge

(c) Contralateral masking. If the stimulus channel routing is changed from L to R or R to L, at the same time, the masking channels routing will be changed from L to R, or R to L to maintain ipsilateral masking.

Single key controls Channel 1 and Shift+key controls Channel 2 unless dedicated key exists for Ch2

# www.myaudiometer.com

● Hearing Assessment ● Fitting Systems ● Balance Assessment

**GN Otometrics, Europe.** +45 45 75 55 55. info@gnotometrics.dk  
**GN Otometrics, North America.** 1-800-289-2150. sales@gnotometrics.com  
**www.otometrics.com**



**otometrics**  
MADSEN · AURICAL · ICS